

**NO MONTHLY FEE**

## Emergency Medical Alert System (model ats4405dl)

**With Telephone, Talking Caller ID And Wireless Pendant**  
**Only \$116.89 + sales tax/shipping**

---



### The Emergency Medical Alert System has it all.

Amplified Telephone, Emergency Dialer, Talking Caller ID, Loud Ringer, Speaker Phone, Wireless Necklace Emergency Pendant.

Feel safer knowing that help is just a button press away!

**Customer Comments:**

"This product had made my life so much easier"  
J. Park, OK

---

"Thank you, thank you, thank you. Living alone and not being able to see very well, this has really made things nice for me. I want to tell everyone how great this is"  
K. Walker, FL

**The Emergency Medical Alert Telephone** is an amplified telephone that will call contacts, neighbors, friends, relatives or emergency personnel with the touch of a button!

The telephone will automatically dial pre-programmed numbers when the button on the pendant is pressed.

The phone has a LOUD RINGER and 25 dB amplification and a 100 Foot Range from pendant to base.

Can be used as a 2 way SPEAKERPHONE. Answer and hang up the speakerphone with the pendant.

#### Feature Details:

- All-in-one telephone, - Adjustable amplified handset, - Tone controls, - Big talking keypad buttons, - Speakerphone,
- LCD display with large characters, - Talking caller ID, - Call waiting caller ID, - Talking phone book, - 64# Caller ID history, - 30# Name and phone directory, - Talking dual language caller ID list, - Hearing aid compatible, - Battery backup (requires 4 AAA batteries, not included), - Voice guide, - Adjustable loud ringer, - Visual ringer alert, - Unanswered call alert, - Real time clock

#### **This system will not be fooled by answering machines!**

How does it work?

When a pendant is pressed the system dials the first number that is programmed into it. The system then waits to 'hear' a zero (0) to be pressed on the other end of the line. This tells the system that a human answered the call. If the systems hears a 0 it will stop calling numbers and turn on the speakerphone. If the system does not hear a 0 pressed it will hang up and call the next number in sequence. We suggest that you have 911 programmed at the end of the call list to ensure that someone is contacted in the event of an emergency.

---

**Assistive Technology Services – [www.ATS-TN.com](http://www.ATS-TN.com) - 1-866-927-3577**