

# Fall Detection - Fall Alert - Man Down Alert

## Medical Alert System - Emergency Alert System

You No Longer Have To Even Press A Button To Get help - NO MONTHLY FEES

---



Systems like these are known by many names: fall detection, fall alert, fall down, man down, movement detection, medical alert with fall detection.

**No Need To Press The Button For HELP.  
If you Fall Down It Automatically Calls for You.**

Great if you are unconscious or black out or can't speak.

Never pay a monitoring fee! No Monthly Fees!

---

### PURCHASE OPTIONS:

---

Brand New with 1 Year Factory Warranty 30 Days To Try And Return If Not Satisfied

---



A total of 2 Waterproof Fall Detection Panic Buttons can be used with this system.



**Retail Price \$319.99**  
**\*\*\* Special ONLY \$299.99**  
**Free Shipping (U.S. Orders Only)**

## Benefits:

- No monthly monitoring charge
- The easiest way to make SOS call - Automatic SOS call
- Dials automatically if the system detects the person has fallen down ( Man Down function)
- No need to "shout" to the base - complete 2 way voice communication right through the panic button.
- Very Small Panic button 2.2" X 1.5" X 0.6"
- Light Weight - it only weighs 2 oz.
- The user can wear the panic button while washing hands or taking a shower (complies with IP-57 waterproof standard)
- Very user friendly - voice prompts for most operations. The system speaks to you guiding you through setup features.
- Persistent contact. This system will continue calling your contacts until someone answers the call or the call is cancelled by the user.
- Can be worn around the neck with a lanyard.
- Optional wrist strap and belt clip is also available.



**Add Extra Waterproof Fall Detection Panic Buttons. Only \$129.99 + shipping**

---

## What's Included:

**AC Power Adapter, Telephone Cord, Waterproof Fall Detect Panic Button (battery included), Base Unit, Instruction Manual, 1 Year Factory Warranty.**

## Features List:

- **2 way communication** through panic button
- Simple operation : push one SOS button on the panic button for emergency calling
- Automatic speakerphone during SOS calling (Talk directly through the panic button)
- Dedicated SOS call flow logic
- Voice prompt for ease of operation
- **Water proof IP-57**
- Low battery alert - Long standby time
- Back up battery (using rechargeable AAA batteries - not included)
- **Automatic fall down detection (to activate Alert Dialing)**
- User can answer any incoming call or make and outgoing call by simply pressing the big button, or cancel the call by pressing the small button
- Priority for SOS call - block all incoming call during SOS mode
- SOS call will continue until it is being answered
- Able to detect answering machine / voice mail from receiver (depends on operator system)
- Voice prompt announcement for most operation status (English, French - selectable by end user)
- BIG SOS panic button to activate telephone call according to preset telephone number
- Ringing Panic Button - when an incoming call is received the Panic Button will ring. This feature can be turned on/off for each Panic Button.
- Pills alarm for reminder to take medicine
- Easy system check to ensure security (this includes battery capacity, power supply, telephone line, distance, out of range etc.)
- Low battery alert, out of range alert, no power and no telephone line alert (Both LED indication and voice announcement)
- Range is about 600 foot range (open area - your environment may effect actual distance)
- Maximum loudness allowing elderly to hear
- Back up battery compartment (4xAAA NiMH battery - not included)
- DECT digital technology provides the best digital and clear voice communication available today
- Each base station can register maximum 2 pendants
- Set up 3 SOS numbers independently (each pendant can dial 3 of their own phone numbers;)

## Easy Installation

- Connect to standard telephone line with the base station (Optional Cell phone usage with Bluetooth adapter if you have no phone line)
- Connect wall power adapter with the base station
- Place the pendant onto base station for charging
- Set up date / time, and maximum 3 SOS telephone numbers
- Extra phone jack in back of unit so you can plug another telephone into this system



# ORDER FORM

**Assistive Technology Services**  
*"Helping You Live Independently"*

8023 Franklin Road  
 Murfreesboro, TN, 37128  
 (615) 562-0043

<b>Billing Address</b> Name _____ Address _____ City _____ State _____ Phone Number _____	Check if same as Billing [ ]	<b>Ship To Address</b> Name _____ Address _____ City _____ State _____ Phone Number _____
---	---	---

CUSTOMER EMAIL ADDRESS		PAYMENT TERMS		DATE
		Due on receipt		

QTY	ITEM #	DESCRIPTION	UNIT PRICE	SHIPPING	UNIT TOTAL
	Atsfd1-us	Fall Detector Medical Alert System Shipped to US	\$299.99		
	Atsfd1-caoth	Fall Detector Medical Alert System Shipped to Canada / Other	\$339.98		
	Atsfd1-xp	Fall Detector Extra Pendant Shipped to US	\$129.99		
	Atsfd1-xp	Fall Detector Extra Pendant Shipped to Canada / Other	\$149.99		
				<b>Subtotal</b>	
		Only Applicable If Shipping To Tennessee		TN Sales Tax 9.75%	
		Prices Subject to Change without Notice		<b>Shipping</b>	
				<b>Total</b>	

<b>Credit Card Type</b>	<b>Credit Card Number</b>	<b>Expiration Month</b>	<b>Expiration Year</b>	<b>Security Code From Back Of Card</b>
Visa [ ] MC [ ] AmEx [ ] Disc [ ]				
<b>Name On Card</b>		<b>Email Address</b>		

"May You Know The Glory Of God And May He Bless You"

If Paying By Check Please Make Checks Payable To: Assistive Technology Services