


# SkyAngel*Beat*

(911 only model)

## User Guide

### Charging:

Use the **Magnetic Charging Cable** to charge the unit for **3 hours** to allow the unit battery to reach 100%. The battery icon on the screen will show the battery as full when the SkyAngelBeat is fully charged. For best results, charge the unit every day. The battery can last up to 72 hours on a full charge.

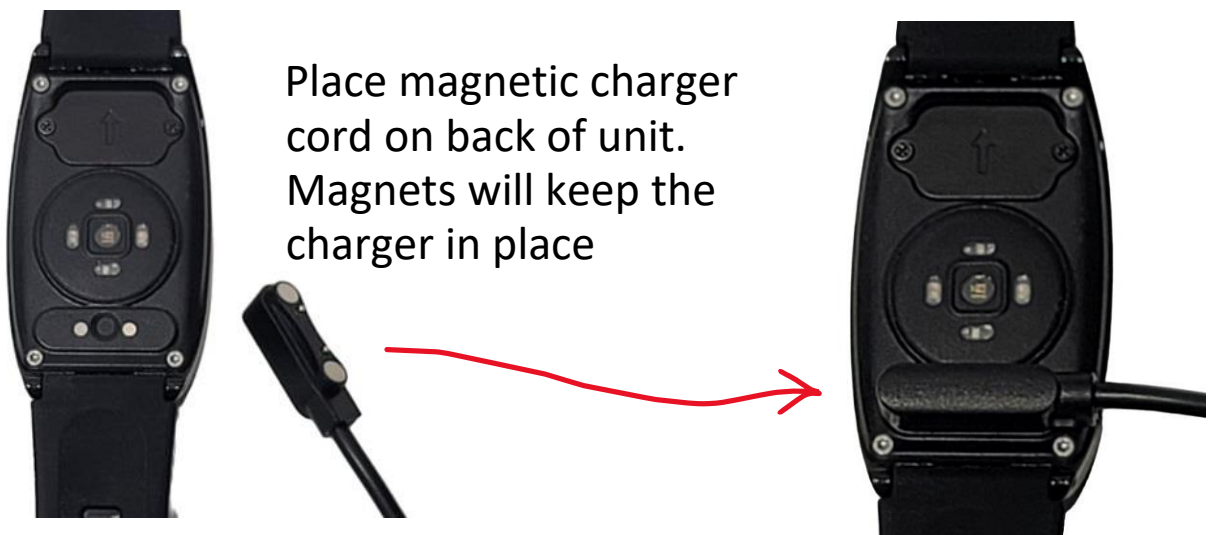

Place magnetic charger cord on back of unit. Magnets will keep the charger in place

**Back of Unit**

**Connected for Charging**



## Heart Button Usage:

- (a):** If unit is off, press and hold button for 5 seconds to power unit on.
- (b):** When powered on, press and hold button for 3 seconds to begin SOS call.
- (c):** If screen is dark, press button briefly to activate display.
- (d):** Press button to scroll through functions.
  
- (e):** During a call, you can press the button to hang up at any time.

## **Activating SOS Alert:**

This device has two ways to activate the SOS function:

### **Press Heart Button:**

When the unit is powered on, press and hold the button for 3 seconds. This will begin a call to 911, after a few moments you will hear 911 through the unit. At this point, you should communicate your address and emergency by speaking to the 911 operator. (If there is no emergency, just let 911 know you are testing the device.)

### **Automatic Fall Detection:**

The unit must be worn so Fall Detection will operate. The unit will automatically call 911 when the fall is detected from 2 feet above the ground. You do not need to press any buttons. The call will connect to 911 automatically so you can confirm your address and type of emergency.

### **Testing fall detection:**

We recommend that you test fall detection upon receiving this unit. Test by placing the unit on your wrist and raising your arm above your head and quickly bring arm down to your side (similar to a jumping jack motion). This should activate the fall detection and you will feel the unit vibrate and make a 'whoop whoop whoop' sound. You can allow the call to go to 911 and tell them you are only testing.

### **Cancellation of an accidental call to 911.**

Simply press the Heart button to cancel a call to 911 before it is answered. If an accidental call is not cancelled before 911 answers, stay on the line and speak to 911 and tell them it was not an emergency call.

## Blood Pressure:

Press Heart button until you see



Wait about 30 seconds to see the results of the measurement



## Temperature:

Press Heart button until you see




Wait about 30 seconds to see the results of the measurement




## Oxygen:

Press Heart button until you see

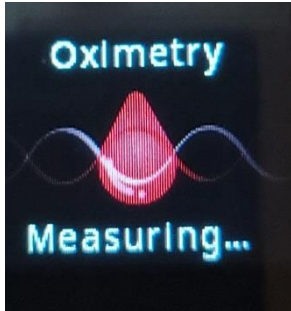

Wait about 30 seconds to see the results of the measurement

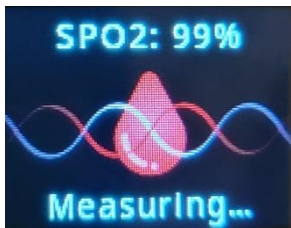

## Power Off:

Press Heart button until you see



Wait about 5 seconds for the unit to turn off

## Date and Time Screen:


Press Heart button until you see



Since the unit does not have a SIM card and there are no monthly fees the watch time and date cannot be set with the free version.

## Message


Press Heart button until you see

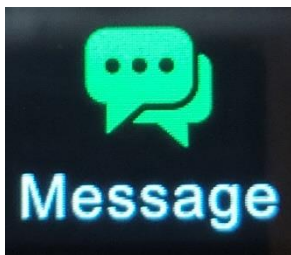

This option is not activated with the Free version.

## Step Counter Screen:

Press Heart button until you see

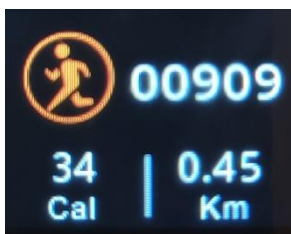

The counter will reset when the unit is turned off.

## Questions:

**Q:** When talking to 911 they do not have a phone number or location.

**A:** This is normal for this device; it does not use a SIM card to call 911 so there is no telephone number or address associated with the unit. The call will still be placed to 911.

**Q:** How can I adjust the Date/Time, Volume, or other settings?

**A:** Since the unit does not have a SIM card and there are no monthly fees these features cannot be changed.

**\*\*Please allow the unit up to 2 minutes to connect to 911 if the cellular network is busy\*\***

This unit can be upgraded to make and receive calls to numbers other than 911 and utilize the tracking and health features. To upgrade your unit to use these features, please call ATS at 615-562-0043 for details.

Scan this QR code with your camera for information on your SkyAngelBeat

