


# SkyAngel*Beat*

*(Friends)*

## User Guide

### Charging:

Use the **Magnetic Charging Cable** to charge the unit for **3 hours** to allow the unit battery to reach 100%. The battery icon on the screen will show the battery as full when the SkyAngelBeat is fully charged. For best results, charge the unit every day. The battery can last up to 72 hours on a full charge.

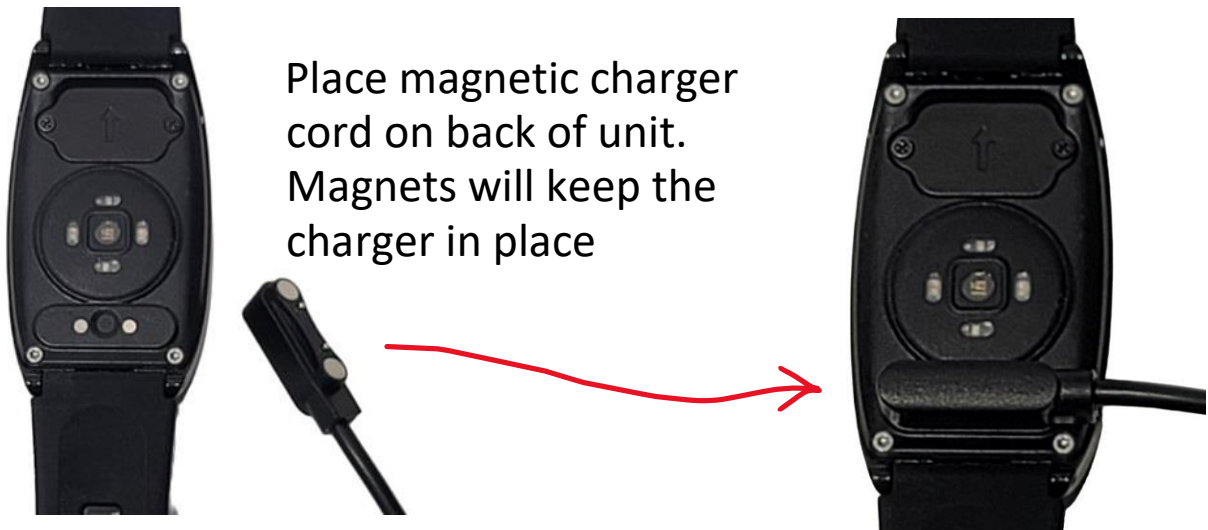

Place magnetic charger cord on back of unit. Magnets will keep the charger in place

**Back of Unit**

**Connected for Charging**



Front of unit

## Heart Button Usage:

- (a):** If unit is off, press and hold button for 5 seconds to power unit on.
- (b):** When powered on, press and hold button for 3 seconds to begin SOS call.
- (c):** If screen is dark, press button briefly to activate display.
- (d):** Press button to scroll through functions.
  
- (e):** During a call, you can press the button to hang up at any time.

## **Activating SOS Alert:**

This device has two ways to activate the SOS function:

### **Press Heart Button:**

When the unit is powered on, press and hold the button for 3 seconds. This will begin a call your emergency contacts, after a few moments you will hear the first contact answer. At this point, you should communicate your emergency by speaking to your contact. If they do not answer within 10-15 seconds the unit will call the next number in order.

### **Automatic Fall Detection:**

The unit must be worn so Fall Detection will operate. The unit will automatically call your emergency contacts when a fall is detected from 2 feet above the ground. You do not need to press any buttons.

### **Testing fall detection:**

We recommend that you test fall detection upon receiving this unit. Test by placing the unit on your wrist and raising your arm above your head and quickly bring arm down to your side (similar to a jumping jack motion). This should activate the fall detection and you will feel the unit vibrate and make a 'whoop whoop whoop' sound. You can allow the call to go to 911 and tell them you are only testing.


### **Cancellation of an accidental call.**

Simply press the Heart button to cancel a call before it is answered.

## Using the Watch:

### Blood Pressure:

Press Heart button until you see

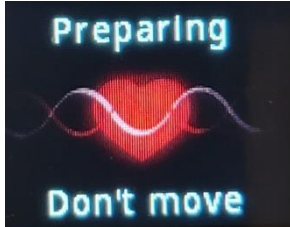

Wait about 30 seconds to see the results of the measurement



### Temperature:

Press Heart button until you see




Wait about 30 seconds to see the results of the measurement




## Oxygen:

Press Heart button until you see

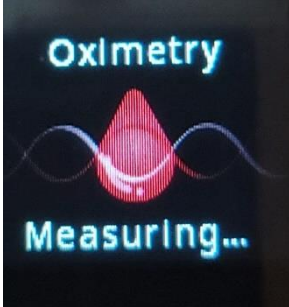

Wait about 30 seconds to see the results of the measurement

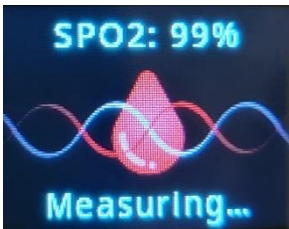

## Power Off:

Press Heart button until you see



Wait about 5 seconds for the unit to turn off


## Date and Time Screen:

Press Heart button until you see



## Message

Press Heart button until you see

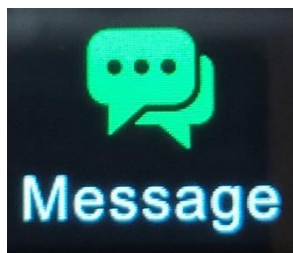

## Step Counter Screen:

Press Heart button until you see



Scan this QR code with your camera for information on your SkyAngelBeat

