

SkyAngelSLIM

PROFESSIONALLY MONITORED

User Guide

****Please charge your unit and call ATS for initial setup and training prior to using your SkyAngelSLIM****

Charging:

Use the **Magnetic Charging Cable** to charge the unit for **3 hours** to allow the unit battery to reach 100%. The battery icon on the screen will show the battery as full when the SkyAngelSLIM is fully charged. The battery can last up to 3 days on a full charge.

For best results, charge every day to ensure the device is always ready to use.



Back of Unit Connect Magnetic Cable for Charging

Feature list:

- Fast Magnetic Charging
- Automatic Fall Detection
- GPS Location
- Calls ATS Professional 24/7 Monitoring Center
- Touch Screen Controls
- Two-Way Voice Communication
- Waterproof – Wear in Shower or Tub

**Right Side Profile: SOS/Power Button:**

- (a):** If unit is off, press and hold button for 4 seconds to power unit on.
- (b):** When powered on, press and hold button for 4 seconds to begin SOS call.
- (c):** If the screen is dark, press button briefly to activate display.
- (d):** Press button to return to home screen from SkyAngelSLIM menu.

Activating SOS Alert:

This device has two ways to activate the SOS function:

SOS/Power Button:

When the unit is powered on, press and hold the SOS button for 4 seconds. This will begin a call to the ATS Monitoring center, after in a few moments you will hear the operator answer the call. At this point, you can communicate through your device.

Fall Alert:

If the fall alert is enabled, the unit will automatically call SOS contacts when the fall is detected. You do not need to press any buttons. The call will be placed automatically.

****Please allow several minutes for the unit to connect a call if the cellular network is busy****

Making a non-SOS call from the SkyAngelSLIM:

With the SkyAngelSLIM, you can make calls that are non-emergency. To do this, follow the steps below:

- Press the **SOS/Power** button to “wake” the screen.
- Swipe to the Left until you see the **Phone** icon
- Tap the **Phone** icon on your home screen
- Enter the number on the on-screen **keypad** you wish to call
- Tap the **Green** phone icon in the bottom right corner to place the call
- Tap the **Red** phone icon to end the call when you are done

Watch Application Overview:

Contacts – not used

Camera – take photos using the camera on the watch

Phone – this allows the user to manually enter a number and make a call

Chat – not used

Qr code- not used

Gallery – this is where the photos are stored

Call log – a list of outgoing calls

Pedometer – counts your daily steps and resets to 0 at midnight automatically

Information- displays device information

Alarm- ability to set daily reminders

Stop watch- ability to measure time

Calculator- general calculator functions

Blood Pressure- ability to measure blood pressure

Pulse- ability to measure your Pulse

Spos- ability to measure Oxygen level

Inactivity- not used

Settings:

Dial Gallery- change the image that is on the watch face. Swipe to change and press and hold to select.

Volume- ability to adjust sound up or down

Brightness- ability to adjust the brightness up or down

Sleep- ability to set the screen sleep time

Volte- not used

Language- ability to change your Language on the watch

Turn Off- this will turn the watch off

Information- displays device information

FAQ:

Q: How do I 'wake up the screen'?

A: Press the Button on the right side on the SkyAngelSLIM.

Q: How can I adjust Screen Brightness, Volume, or other settings?

A: On the SkyAngelSLIM swipe to the '**Settings**' the available options will be listed in this menu.

Q: How can I turn the SkyAngelSLIM off?

A: From the SkyAngelSLIM main screen swipe 3 times to the left to go to "**Settings**" and scroll down to "**Turn off**". Tap the "**Turn off**" icon and tap the Green "OK" check symbol.

Q: How do I see battery status and signal status on the SkyAngelSLIM?

A: From the SkyAngelSLIM screen press the side button if the screen is sleeping. Swipe from the top down to see the status screen.



Scan this QR code with your camera for information on your SkyAngelSLIM